

Using a Te Pari Dosing Gun with a 5000 indicator

This document shows how to connect and use a Te Pari Revolution Dosing Gun with a 5000 series weigh scale indicator. This setup guide applies to the XR5000, ID5000 and JR5000 indicators, although the instructions are slightly different for each model.



The Te Pari Dosing Gun connects to the 5000 indicator via a phone app. The instructions provided here are for Android phones only. iPhones are not supported.

Before you start

- Update the 5000 indicator firmware to Version 4.3 or above, using Data Link PC Software.
- Download the Te Pari Dosing Gun app from the Google Play Store.
- Ensure that the Wi-Fi and *Bluetooth*[™] settings are enabled on your Android phone.
- Ensure that the *Bluetooth* setting is enabled on your 5000 indicator.

Setting up the Te Pari Revolution Dosing Gun

1. On the 5000 indicator, start a new session:



2. Check that the indicator is receiving weights by putting a test weight on the scales:



3. On the Dosing Gun, press and navigate to the *Wi-Fi* menu. Enable **Wi-Fi**:





5. Press again to access the *DG SSID* menu. Use the arrow keys to select **Te Pari**:



6. Press 🖬 again to access the *Wi-Fi Security* menu. Use the arrow keys to select **X**:



Setting up the Te Pari Dosing Gun app

- 1. Make sure that the 5000 indicator and the Dosing Gun are both ON.
- 2. If connecting to the 5000 indicator for the first time, go to the *Bluetooth* settings on your Android phone and pair to the 5000 indicator:

< Bluetooth	Stop :				
On					
Make sure the device you want to connect to is in pairing mode. Your phone (Galaxy S10) is currently visible to nearby devices.					
Paned devices					
□ XR5000	ැබූ				
Available devices					

3. Go to the *Wi-Fi* settings on your Android phone and connect to the **TePari** network.

< V	Vi-Fi	» :
On		
Curren	t network TePari Connected without internet	③ 段

- 4. Launch the app on the Android phone.
- 5. Go into the *Settings* menu and select Tru-Test.



6. Tap New Session. Complete the fields to set the dosing, then tap Save & Start:

3:20 🔅 📞	জনা 52% <u>⊜</u> _	2:34 🖭 🏟 🕅 🔸	
		← New Session	
TE PARI Nižkus		Dosing type	
		Oral Drench	
		O Pour On	
		Injection	
		Product Name	
New Session		Product	
		Product Batch Number	er
		Batch number	
Result	e Session	Value of Drench per L	itro
🕓 History		Drench per litre	
(Connec	cted Devices	Dose Type*	
Setting	s	Fixed Dose Calculated Do	200
			100
😟 Help		Application Rate Dose*	Weight*
😥 Upgrad	e Drench Gun Firmware	E.g. 20mL	per E.g. 100Kg
			CANCEL SAVE & START
			SAVE & START

7. Tap **Connect**, then **Weigh Scale** then tap on the indicator e.g. **XR5000**.

	?	Drench Gun Connected	
Connect Device	*	Weigh Scale Connect	S21 C Ref 51%€
App is not connecting to the drench or/ and the scale, please connect it.	*	EID Reader Connect APP SETTINGS CLOSE	Select a device to connect XR5000 00:25:CA:6A:7B:A6

The app screen will now show that the 5000 indicator is connected. Tap **Close**.



The Dosing Gun will display with a small Wi-Fi connected symbol in the top, left corner.



8. Now, when an animal steps onto the scale, the Dosing Gun will display the dose. Pull the trigger to deliver the dose. The app will display both the animal weight and the dose:



To record animal IDs on the Dosing Gun during the session, you can either enter a VID manually in the app or connect an EID Reader to the app and scan animal EIDs as they are weighed and treated.

Contact the Datamars Livestock Smart Farming Support team:

sf-support@datamars.com AU 1800 248 774 NZ 0800 248 722 US +1 (888) 431-0957 (toll free) UK +44 20 4571 7717 IE and EU +353 (21) 242 8844

> © Datamars Limited, 2023. All rights reserved. Issue 1 11/2023